

INSTRUCTIONS FOR USE

VISMED® – Sodium hyaluronate 0.18 %.

Lubricant eye drops.

Sterile, free from preservatives.

Composition:

Each 1 ml of solution contains: sodium hyaluronate 1.8 mg, sodium chloride, potassium chloride, disodium-hydrogenphosphate, sodium citrate, magnesium chloride, calcium chloride and water for injections. The solution is hypotonic.

Intended use:

Moderate or severe sensations of dryness in the eye.

Contra-Indications:


Individual hypersensitivity to any constituent of the product.

Interactions:

Do not use VISMED® at the same time with any drug or other product applied to the eye since it may modify their effects.


Directions for use:

Twist off tab. If not otherwise recommended place one drop of VISMED® into the conjunctival sac of each eye three times a day or as often as needed. After blinking, the solution will disperse and form a transparent and long lasting coating on the surface of the eye.

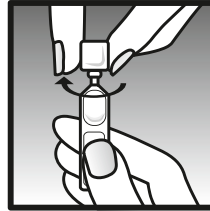

Twist off cap.

1

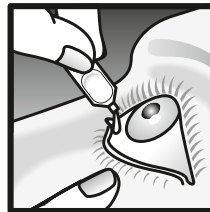

Instill solution into eye.

2



Do not reuse!

3



Precaution for use:

Do not touch the tip of the container and do not touch the surface of the eye with the tip of the container. Do not use VISMED® if the container is damaged. As VISMED® does not contain preservatives, any solution not used immediately after opening should be discarded.

Store between 2 °C and 25 °C! Do not use VISMED® after the expiry date indicated on the package. If discomfort persists while using VISMED® or reappears after discontinuation of administration, consult a physician.

Keep out of reach of children!

Characteristics and mode of action:

VISMED® contains sodium hyaluronate, a natural polymer, which is also present in the structures of the human eye. The particular physical characteristics of sodium hyaluronate confer to VISMED® its "viscoelastic" and water retaining properties. VISMED® provides a stable coating on the surface of the eye which is only slowly eliminated by blinking. Therefore VISMED® combines long lasting relief with maximum comfort. VISMED® is well tolerated due to its unique composition. It is free from preservatives.

Presentation:

20 monodoses units of 0.3 ml.

Last revision date: March 2008



Batch number



Expiry date



For single use only!



Refer to instruction leaflet



6106222/1
S/03.08



Manufacturer: TRB CHEMEDICA AG
85540 Haar/München, Germany

Distributor: TRB CHEMEDICA (UK) Ltd., MED IC3
Keele University Science Park
Keele, Staffs ST5 5NP, United Kingdom